

# Sickle Micropipettes

## INSTRUCTIONS FOR USE

### INTENDED USE

Disposable pipets are used to collect and dispense a preset 20µl sample to be used with *SICKLEDEX*® test kits.

### SUMMARY AND PRINCIPLES

*SICKLEDEX* test kits require a 20µl sample. Disposable pipets are a convenient way to obtain the sample and are compatible with all configurations of *SICKLEDEX* test kits.

### PRECAUTIONS

1. For In Vitro Diagnostic Use.
2. All patient blood samples should be treated as if they are capable of transmitting infections and should be handled with appropriate precautions. Avoid contact with skin and mucous membranes.
3. Used pipets containing blood should not be disposed of in general waste, but should be disposed with infectious medical waste.

### INSTRUCTIONS FOR USE

Never squeeze the plastic tube while sampling. Filling is automatic.

1. Hold the tube horizontally, and touch the tip of the pipet to the sample. Capillary action will automatically draw the sample to the fill line and stop.
2. To expel the sample, align the tip of the pipet with the sample tube and squeeze the bulb. If a sample won't expel, verify that the pipet has been filled all the way. If necessary, touch the tip to the sample again and allow it to fill completely. Then align the tip with the sample tube and squeeze the bulb to deliver the sample.
3. Expel the sample within 30 seconds of collection to avoid coagulation in the micropipette.

### LIMITATIONS

1. For Single Use Only.

### EXPECTED RESULTS

When the disposable pipet is used appropriately it will collect and dispense a preset 20µl sample.

### ORDERING INFORMATION

Please call our Customer Service Department at 800-228-6090 for assistance. Additional information can be found online at [streck.com](http://streck.com).

### GLOSSARY OF SYMBOLS

See the Instructions (IFU) tab under Resources on the product page at [streck.com](http://streck.com).

See [streck.com/patents](http://streck.com/patents) for patents that may be applicable to this product.



Streck  
7002 S. 109 Street, La Vista, NE 68128 USA

880026-4  
2020-05